


Processing Urine Samples with BD Vacutainer® Collection Products

Transfer Straw and C&S Preservative Tube



1. Submerge tip of transfer straw in specimen.

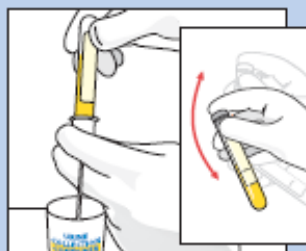

2. • Push C&S Preservative Tube (gray top) into the transfer straw all the way.
 - Hold in position until flow stops.
 - Remove tube, leaving transfer straw in specimen container.
 - Shake tube vigorously.



3. Label filled tube with patient's name, the date/time of specimen collection and any other data required by your institution.



4. Dispose of transfer straw in sharps collector.

BD Global Technical Services: 1.800.268.5430

BD Customer Service: 1.866.979.9408

www.bd.com/ca

Handle all biologic samples and medical "sharps" (lancets, needles, and transfer straws) according to the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biologic samples (for example, through a puncture injury) since they may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. BD does not recommend resheathing used needles, but the policies and procedures of your facility may differ and must always be followed. Discard any medical "sharps" in biohazard containers approved for their disposal.



2771 Bristol Circle
Oakville, ON L6H 6R5